

Half-Face Reusable Respirators/Cartridge Infographic



User seal checks

Half-face reusable respirators | cartridge

NEGATIVE AND POSITIVE PRESSURE



Breathe **in** gently

Respirator should
slightly **collapse**

If air leaks **in**
around the face:



Adjust straps or
Reposition face piece



Breathe **out** gently

Respirator should
slightly **expand**

If air leaks **out**
around the face:



REPEAT TEST UNTIL SEAL IS ACHIEVED

 manufacturing
Safety Alliance of BC

Helping you *Make it Safe*
safetyalliancebc.ca

Source: safetyalliancebc.ca/covid-19